

SHR Pain-free hair removal

Spa Accord delivers permanent hair reduction and long-term hair removal with the same revolutionary SHR technology found in medical clinics. The system safely and gently heats the dermis till the hair follicle is damaged, preventing regrowth, yet does not injure the surrounding skin or cause client discomfort.

Alma Lasers IN-Motion™ technology

Spa Accord is pain-free because its unique laser design allows it to be applied in motion, using the SHR sweeping technique that prevents missed or skipped spots. The comprehensive coverage means smooth legs, arms, backs and faces for all your customers—without the pain. In fact, some compare the experience to a soothing hot stone massage.

Safe for every customer, every time

With Spa Accord™ you can offer pain-free hair removal all year round. All skin types—including tanned skin—and hair colors will benefit.



How Spa Accord works

The 810-nm diode laser is configured for safe, Alma Lasers IN-Motion operation.


For the first time, this laser, which is the gold standard of the medical aesthetic community, is available for aestheticians and cosmeticians.

Gradual heating is the key


Low, rather than high, fluence at high average power is what makes Spa Accord uniquely comfortable and effective. Unlike other systems that blast away unwanted hair, Spa Accord heats the hair follicles very gradually, so a pain sensation is never triggered. In fact, this approach is so painless that Spa Accord does not even require very cold gel.

Fast coverage for shorter appointments


In fact, Spa Accord has the fastest coverage rate in the industry at a 10 Hz pulse repetition rate. That means you can treat customers more quickly, increase appointment bookings and see the return on your investment even faster.

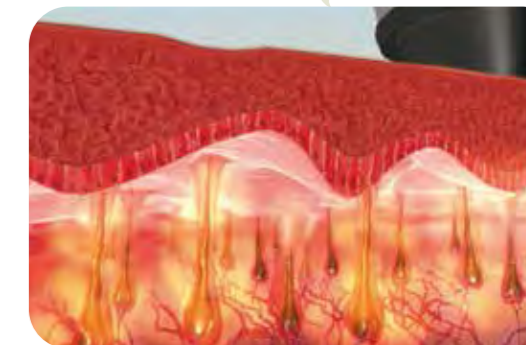

During a treatment

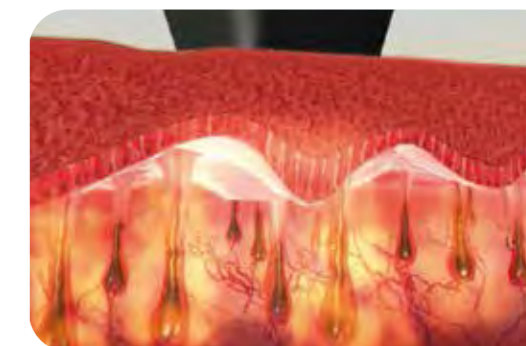

During a treatment

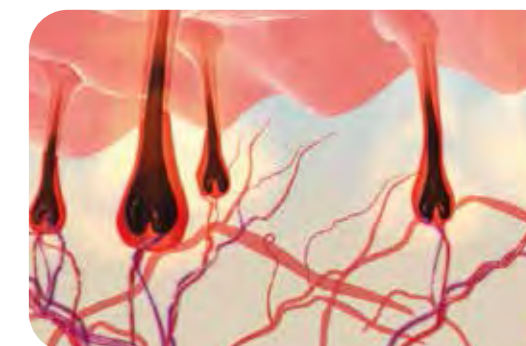

After a treatment

IN-Motion™ technology ensures comfort

IN-Motion™ technology represents a breakthrough in customer comfort, speed of procedure, and repeatable clinical results. It combines cooling with a gradual thermal rise to the target therapeutic temperature, without the risk of injury and with much less pain.




Large 8" color touch screen includes state-of-the-art interactive software that ensures a safe and pleasant experience for clients. Easy-to-follow instructions make it foolproof for the spa.



Spa Accord screen for bikini line treatment



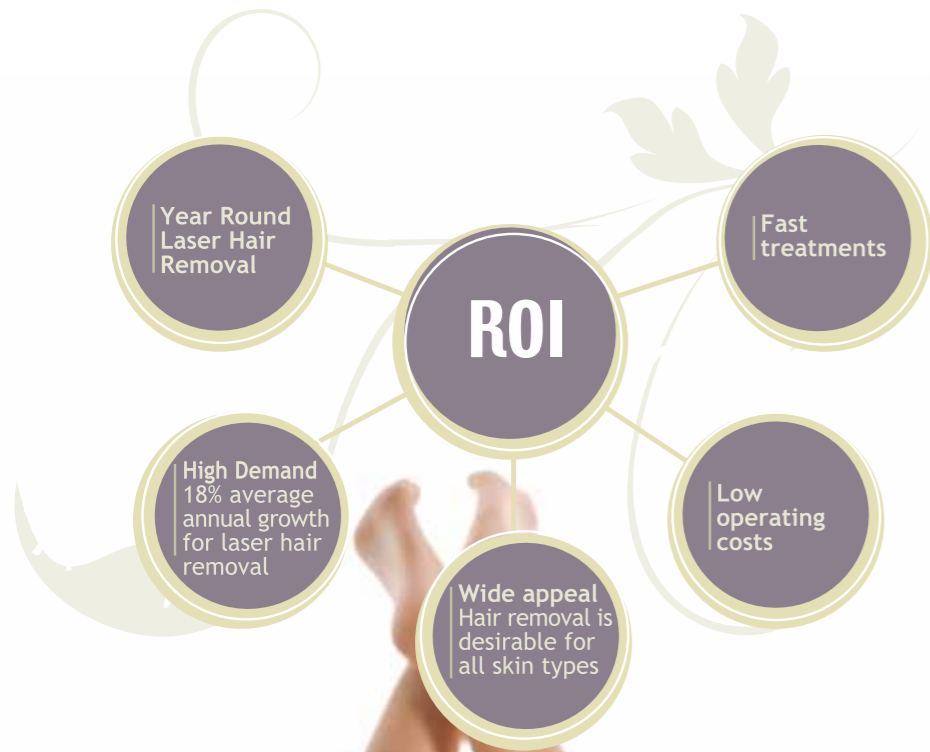


A growing segment: male hair removal

Permanent hair removal no longer appeals exclusively to women. Many men have also started to seek long-term hair reduction solutions, and Spa Accord is particularly well-suited for men because the treatment is quick, shows rapid results, and is pain-free.

Less expensive than waxing

With this unique combination of comfort, safety and effectiveness, Spa Accord appeals to a wide range of spa clientele. In fact, your customers may be surprised to learn that permanent laser hair removal treatments may actually be less expensive overall than repetitive waxing or other options.

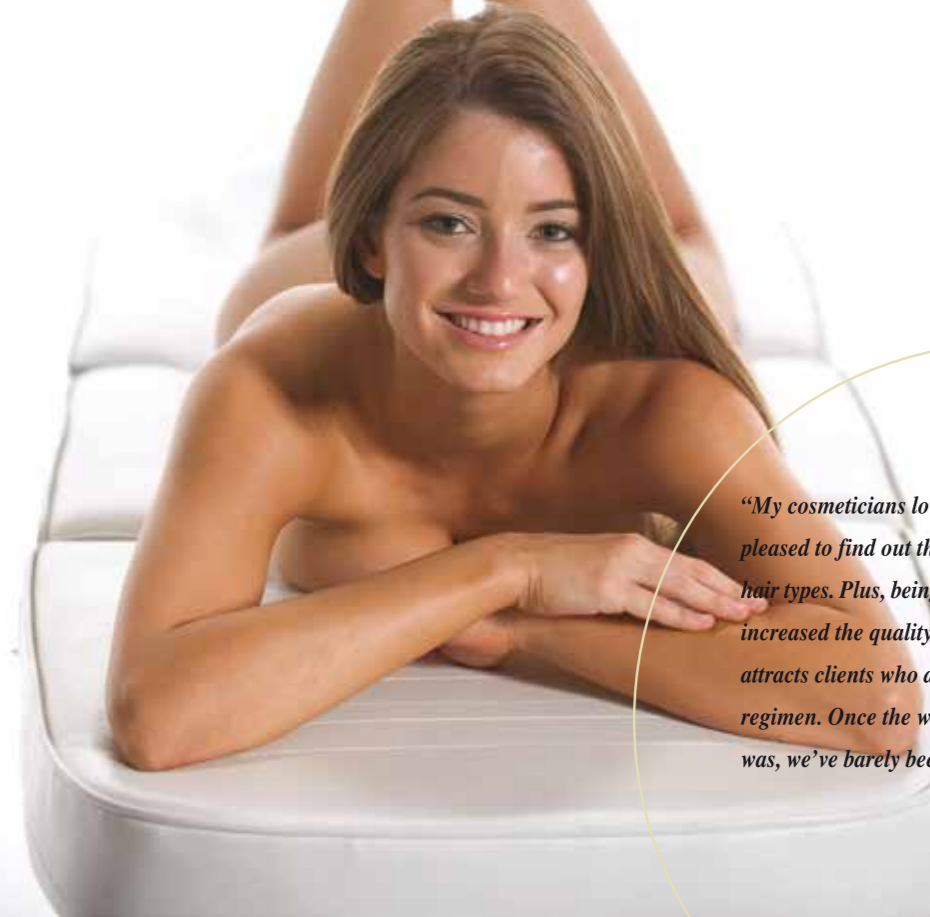

“My cosmeticians love how easy it is to operate. We were pleased to find out the system was foolproof for all skin and hair types. Plus, being able to offer Spa Accord treatments has increased the quality and quantity of our spa’s clientele. It attracts clients who are willing to spend more on their beauty regimen. Once the word went out about how truly painless it was, we’ve barely been able to keep up with all the inquiries.”


Spa Esprit



The Spa Accord difference

- Pain-Free Hair-Free laser hair removal
- Permanent hair reduction for all skin types, including tanned clients
- Treatment availability all year round
- Low cost of ownership with no disposables (tips, pulses)
- Fastest coverage rate in the industry
- Safety assured through Interactive software designed for cosmetician / spa use

Before

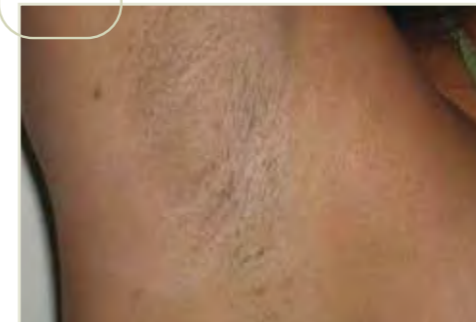

After



Photos Courtesy: Dr. Michael Shohat

spa Accord

Introducing a medically-proven system for permanent reduction of unwanted facial and body hair—without pain. Spa Accord™ is an innovative laser device, specially designed for cosmeticians and beauticians.

Spa Accord represents decades of experience in laser hair removal by leading physicians that have been distilled into a simple, easy-to-use system for permanent, painless hair removal.

If you currently offer waxing, threading or other repetitive, painful hair removal techniques, now you can provide your customers luxurious, pain-free Spa Accord treatment.

Outlet power required	Up to 10 J/cm ²
Light source	Diode laser (810 nm)
PRR	10 Hz
Cooling	Integrated cooling system
Display	Interactive 8" color touch screen
Electrical	120/230 VAC, 13/6 A, 50/60 Hz
System dimensions	14" x 16" x 43.3" (35 x 40 x 110 cm)
Weights	139 lbs (63 kg)

Alma Lasers™
Wellbeing Through Technology

www.almabeauty.com

Spa Accord is manufactured by Alma Lasers, Ltd. a global developer, manufacturer and provider of laser, light-based, radiofrequency and Ultrasound devices for aesthetic, medical and spa applications.

US Headquarters
485 Half Day Road
Suite 100
Buffalo Grove, IL 60089
Tel +1-224 377-2000
Fax +1-224 377-2050
marketing@almalasers.com

International Headquarters
14 Halamish st.
Caesarea Industrial Park
Caesarea 38900, Israel
Tel +972-4-627-5357
Fax +972-4-627-5368
info@almalasers.com

CE



PBSPA09021001




IN-Motion
Pain-Free
Hair-Free



Long
Lasting Hair
Removal

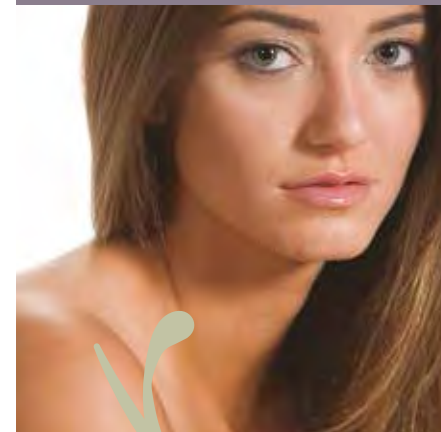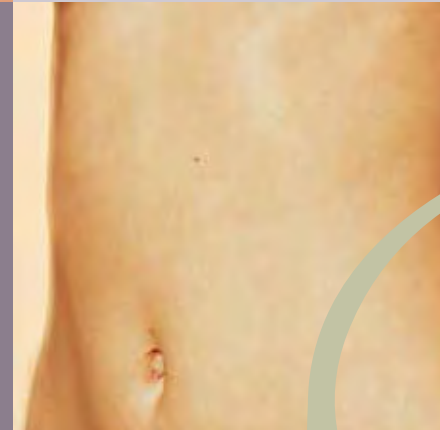

Year Round
Laser Hair
Removal

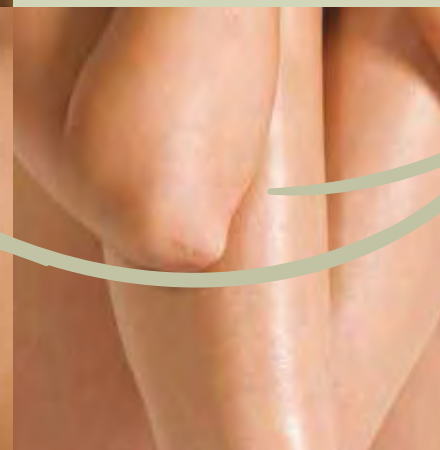

SHR
Diode Laser
Technology

IN-Motion
Pain-Free
Effective



Virtually Pain free Hair Removal